

SimplyGo, ready for a stroll

Quick start guide



Why SimplyGo?



Hello, my name is

I am suffering from COPD.¹

My physician has prescribed for me
long-term oxygen therapy that requires
the regular use of high concentrations
of oxygen. My SimplyGo portable oxygen
concentrator provides me with this

reliable supply of oxygen. It is important for me to use oxygen therapy² during both my daily activities and at night. My SimplyGo stays with me wherever I go and helps me get the exercise I need. I can walk, stand, move about, and basically enjoy my day while I receive my daily oxygen therapy. It can improve the quality of my life.

¹COPD (Chronic Obstructive Pulmonary Disease) refers to a group of lung diseases, such as emphysema and chronic bronchitis, that obstruct air from easily entering and exiting the lungs, thus preventing the proper elimination of carbon dioxide.

²Portable oxygen therapy: use and benefit in hypoxaemic COPD patients on long-term oxygen therapy – J. Vergeret, C. Brambilla, L. Mounier – ERJ 1989

SimplyGo - Getting started



Select the mode you've been prescribed



Pulse mode SimplyGo detects each breath intake and delivers the oxygen delivery accordingly.




Continuous-flow mode Continuous oxygen delivery from 0.5 to 2 LPM.



Sleep mode Pulse mode with an increased detection sensitivity to sense breath intake more accurately at night. SimplyGo automatically switches to the last continuous flow setting used if no breath is detected after a period of time.

Operating SimplyGo



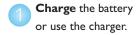



Insert SimplyGo into its carrying case.



Insert the charged battery.



Connect a standard single lumen nasal cannula. Make sure it is not

pinched or kinked. Maximum length up to 9 m (30 ft).



Turn on/off SimplyGo by pressing the power button twice.



Select the mode by pressing the mode (M) button. Continue pressing the Mode button to scroll through the Pulse, Continuous flow, or Sleep modes.



Set the flow rate in accordance with your oxygen prescription using the plus (+) or minus (-) buttons.



Make sure you have all the accessories you

need. Install SimplyGo on the cart.



Position the nasal cannula on your face and breathe normally through the nose.

Simply before you go

SimplyGo is your ideal travel companion. Discover some useful tips when traveling with a portable oxygen concentrator.

Inform your physician about your trip; your prescription may need to be adapted depending on the climate or altitude of your destination.

When booking your trip, inform the transportation company that you intend to use your SimplyGo during the journey.

Learn about the carrier's specific policy or requirements regarding oxygen therapy patients.

Make sure you have all the SimplyGo accessories you need. Contact your homecare provider if necessary.

At departure

Make sure you carry a copy of your oxygen therapy prescription with you at all times. Make sure you have the name and contact information of your homecare provider. Copy these onto your SimplyGo Quick start guide.

Verify that your SimplyGo batteries are fully charged and that you have your oxygen cannula with you.

Verify that you have the necessary power supplies to recharge SimplyGo batteries.

Keep your SimplyGo Quick start guide and your SimplyGo user manual together with your device at all times.

SimplyGo on the plane

The United States Federal Aviation Administration (FAA) has approved the SimplyGo for in-flight use on commercial airliners.

Although the FAA and other international aviation regulatory bodies allow the use of approved portable oxygen concentrators on commercial airlines, many domestic and international carriers have additional requirements including advance notification, and possession of a physician's letter and oxygen prescription.

If you are planning to fly with your SimplyGo, contact your airline well in advance of travel.

SimplyGo battery specification

Battery autonomy	
□ Pulse mode setting 2*	3 hr
	54 min

^{*}Based on 20 breaths per minute

Charge time: 2-3 hours per battery when fully discharged



Ready. Set. SimplyGo!

My provider

Company Address	
Address	
Tel	
My contact name	
Mobile phone	
e-mail	

My prescription

Patient	
Physician	
Phone	
e-mail	
My prescription	

Philips Healthcare is part of Royal Philips

How to reach us

+49 7031 463 2254

Latin America +55 11 2125 0744

North America

+1 425 487 7000

www.philips.com/healthcare Philips Respironics
healthcare@philips.com 1010 Murry Ridge Lane

Murrysville, PA 15668

Asia

+49 7031 463 2254 Customer Service

+1 724 387 4000

Europe, Middle East, Africa 800 345 6443 (toll free, US only)

Philips Respironics International Headquarters

+33 1 47 28 30 82

Philips Respironics Asia Pacific

+65 6882 5282

800 285 5585 (toll free, US only) Philips Respironics Australia

+61 (2) 9947 0440

1300 766 488 (toll free, Australia only)

www.philips.com/respironics

Respironics and SimplyGo are trademarks of Respironics, Inc. and its affiliates. All rights reserved.

Please visit www.philips.com/ReadySetSimplyGo



© 2013 Koninklijke Philips N.V. All rights reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

Broudy AE 6/21/13 MCI 4105712 PN 1098671